

Innovative Products That Improve Patient Care



An easy to use bed bath system that is efficient,
effectively hygienic and can be used anywhere at anytime.
Everything required is in the bag.

Distributed By

 **KENERIC**
HEALTHCARE
www.KenericHealthcare.com

8925 Sterling Street, Suite 100 • Irving, TX 75063, USA • Tel +1 855 872 2013

PUREPURGE™

Pre-Packaged Bed Bath System



pH Balanced and Infused with Vitamin E & Aloe to Ensure Clean and Conditioned Skin



 **KENERIC**
HEALTHCARE
Innovative Products That Improve Patient Care

www.KenericHealthcare.com



PUREPURGE™ Bed Bath System

PurePurge™ Bed Bath System is for patients who are unable to bathe in a tub or shower. It is an easy to use system that is efficient, effectively hygienic and can be used anywhere at anytime. Everything required is in the bag.

PERFECT HYGIENE

- » Gentle needlepunch fabric towels contain just the right amount of PurePurge™ advanced cleansing formula to thoroughly clean and condition the skin.
- » Hair is left glistening and conditioned.
- » Time saving with no rinsing and no need to towel dry.

REMOVES RISK

- » PurePurge™ Bed Bath eliminates the cross-infection risks of dirty basins, contaminated water supplies, and the skin damaging effects of harsh soaps and rough laundered terry towels.
- » No more carrying of heavy water filled basins.
- » 1 towel for each of the 8 body zones to help control cross contamination across the body.

COST EFFECTIVE

- » PurePurge™ Bed Bath process can be completed in just 10 minutes, compared to traditional methods that can take up to 30 minutes.
- » No clean up time - no storing of basins, towels, soaps and lotions.
- » No laundering of terry towels.

CONSISTENCY OF PROCEDURE

- » PurePurge™ Bed Bath System means process efficiency and consistency throughout the facility.
- » Eliminates any inconsistency in the type of soap, shampoo and skin conditioners that could be used across the facility.

BALANCED pH

- » Balanced pH ensures a skin-friendly experience for patients.
- » Improved skin condition provides an important barrier to infection.

PARABEN AND LATEX FREE


- » PurePurge™ Bed Bath System provides a dignified bathing experience that leaves the patient clean and comfortably refreshed.



PUREPURGE™ BED BATH WIPES

Discard each used towel into an appropriate waste receptacle
KHC31010000


- » 8 medium weight needlepunch towels
- » Warm pack in microwave if required – 15 to 20 seconds 800w
- » Check if temperature is OK before using
- » Remove and use towels one at a time - 1 towel for each body zone
- » Commence at body zone 1 and end at body zone 8
- » Reseal pack after each towel to maintain temperature inside pack
- » Convenient resealable pack keeps unused towels in good and moist condition

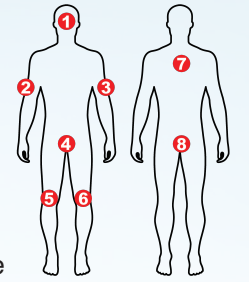

PUREPURGE™ BED BATH GLOVES

Discard each used glove into an appropriate waste receptacle
KHC31020000

- » 8 easy-to-don mitt style gloves
- » Warm pack in microwave if required – 15 to 20 seconds 800w
- » Check if temperature is OK before using
- » Remove and use gloves one at a time - 1 glove for each body zone
- » Commence at body zone 1 and end at body zone 8
- » Reseal pack after each glove to maintain temperature inside pack
- » Convenient resealable pack keeps unused gloves in good and moist condition

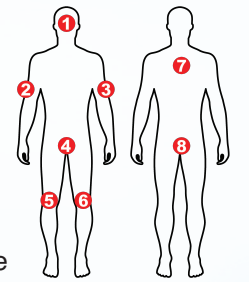

PUREPURGE™ SHAMPOO CAP

Discard each used cap into an appropriate waste receptacle
KHC31030000

- » 1 easy to use shampoo cap
- » Warm pack in microwave if required – 15 to 20 seconds 800w
- » Check if temperature is OK before using
- » Remove cap from pack and place on patient's head, massaging for 3 minutes
- » Towel dry if necessary and comb through hair as required

PRODUCT	PACK CONTENT	PACK REF	CASE OF 30 PACKS
Bed Bath Wipes	8 Wipes	KHC31010000	KHC32010000
Bed Bath Gloves	8 Gloves	KHC31020000	KHC32020000
Shampoo Cap	1 Cap	KHC31030000	KHC32030000