

DESCRIPTION:

RTD™ Wound Dressing is made of polyurethane/polyether foam, Methylene Blue (0.25 mg/g), Gentian Violet (0.25 mg/g), and Silver Sodium Zirconium Phosphate (7 mg/g).

INDICATIONS:

The RTD™ Wound Dressing is designed for use only in adults and for the management of partial to full thickness wounds with light to heavy exudate, and has demonstrated effectiveness for pressure ulcers, venous stasis ulcers, and diabetic foot ulcers.

APPLICATION: Easy as 1-2-3

1. Measure and Cut RTD™ Wound Dressing to the size of the wound. RTD Wound Dressing can be rolled or layered and packed into a wound.
2. Apply RTD™ Wound Dressing in direct contact with the wound bed. Direct application is necessary for best results.
3. Cover with non-occlusive secondary dressing of choice. Leave in place for up to 3 days.

CONTRAINDICATIONS

RTD™ Wound Dressings are not indicated for burns; pregnant or lactating women; or children.

PRECAUTIONS

In the event of any clinical infection, RTD™ Wound Dressing is not intended to replace systemic therapy or appropriate course of treatment for infection.

When choosing a wound dressing, clinicians should consider any potential adverse effects for any patient.

Discontinue use if allergic reaction is suspected.

Caution:

Rx only. Federal Laws restricts this device to be sold by or on the order of a licensed physician.

RECOMMENDATIONS

Wound Preparation: Prior to applying RTD™ Wound Dressing, clean the wound with an appropriate cleaning agent per institution's procedures.

Standard precautions should be used to prevent spread of infectious agents when dressing wounds.

Securing RTD™ Wound Dressing In Place:

Use gauze®, Kerlix®, Omnifix®, Mefix®, other non-occlusive adhesives or medical grade tape.

Tips: For best results, RTD™ Wound Dressing should be in direct contact with the wound bed and fit to the edges of the wound.

For moderate to heavy exuding wounds the 1/4 inch thickness is recommended. For light exuding and/or superficial wounds the 1/8 inch thickness is recommended.

Leave RTD™ dressing in place for three days or remove if saturated. If the dressing becomes saturated before the desired dressing change, a second dressing may be placed on top for continued absorption. If The RTD™ dressing is difficult to remove, soak with saline or sterile water. A non-adherent layer is recommended to prevent sticking or in-growth at the later stages of wound healing.

In Home Use: RTD™ Wound Dressing should be used under the direct supervision of a wound care professional.

STERILE R

Rx Only



Do not use if package is opened or damaged



See Instructions



Single use only



Keep in dry place.



Avoid storing at higher temperatures 44°C/ 112°F



0044

EC REP

Obelis s.a., Bld.
General Wahis 53,
1030 Brussels, Belgium